

TTAF® Instructions for Ultraviolet Disinfection Lamp

Thank you for purchasing our products, in order to ensure that you can fully understand and install this product, please read the instructions carefully before using it for the first time. Please keep the book after reading it for future reference.

Product specification

| Light source | Power | Applicable area | Operating Method | Body Material | Other Features |
|--------------------|-----------|--------------------|------------------|---------------|---|
| Ultraviolet source | 25W~35W * | <=30m ² | Touch control | Anti-UV PC | Human Body Sensing 3 Operating modes for timing Delayed operation to leave the room |

* This manual is suitable for 25W ~ 35W UV disinfection lamp products

Warning


1. Please strictly refer to the operation instruction.
2. Do not dismantle the machine or replace any parts on your own.
3. In the process of disinfection, please be sure to evacuate human, animals and plants from the space to avoid radiation.
4. Please do not stare at the light when it is working to avoid damage to your eyes.
5. Do not hit or fall over the product, or it may damage the product or lamp tube.
6. If you accidentally break the lamp tube, please pay attention to keep the scene ventilated and clean debris in time.
7. This product is an electrical product, pls pay close attention to electricity safety.

Caution:

- If the power on but the light doesn't work, please confirm whether the power plug is properly connected with the power source.
- ※ Please select the correct input voltage for this product.
 - ※ Please do not use the product in unstable, flammable or humid places.
 - ※ To ensure safety, please have electrical professionals to repair or inspect the product.
 - ※ In case of abnormal phenomenon of this product, please cut off the power supply immediately, and ask electrical professionals to repair it.
 - ※ The product cannot be used on slopes larger than 6° or in unstable places.
 - ※ Do not place the high voltage AC plug in the damp or wet place to prevent electric shock.




Part name and method of use

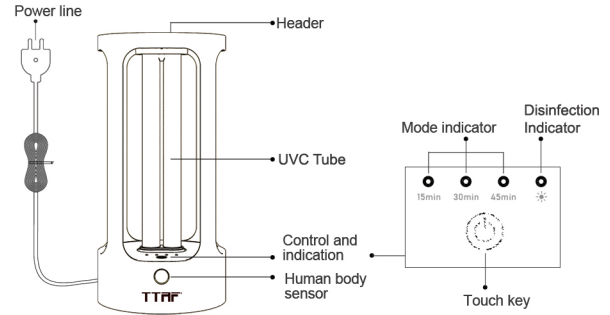


How to Use

1. Insert the plug connected to the cable of the device into the socket.

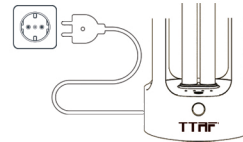


2. Select the disinfection time

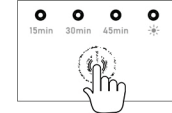

Click the button to select the disinfection time

- ※ Suggestions for gear selection
- 15min: apply to 10m²
- 30min: apply to 20m²
- 45min: apply to 30m²

3. Determine disinfection time with short presses on the button

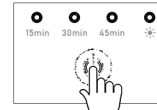

After the timing is determined, press the button until the buzzer beeps (About 3 Seconds). Disinfection Indicator will start flashing.

4. Start Disinfection:
People should leave the room within 20 seconds. The disinfection process will start automatically.
5. Disinfection process:
No human or pet should enter the disinfected room during the operation.

