



MINI STERILIZATION BOX
DB-060-B-UVC
USER MANUAL



Basic Characteristics

- UV-C Lamp Sterilization Bag provides effective sterilization thanks to UV-C rays.
- Sterilization time is 3 minutes.
- Provides sterilization without radiation, ozone, water and chemicals.
- It sterilizes all surfaces of your belongings with UV-C Lamp and reflectors providing 360 ° coverage.
- Thanks to its portable design, it can be used at home, office, hospitals, workplaces, in hotels etc. ideal for use.

Technical Specifications

Product Code	: DB-060-B-UVC
Adapter Input Value	: 100~240VAC 50/60Hz
Adapter Output Value	: 5V DC 1,5A
Power Consumption	: 10W
Source of Light	: UV-C LED
UV Ray Wavelength	: 265~285nm
Working Time	: 180 sec.
Inner Dimensions	: 320x250x50mm
External Dimensions	: 335x265x110mm
Weight	: 1600g

Package Included

- I. Sterilization Bag
- II. Adapter
- III. UV Protective Glasses

Operation Steps

1. Plug in the adapter of the device.
2. Open the cover of the device and place the product you want to sterilize into the device.
3. Press the on-off button on the device.
4. The sterilization process will end after 3 minutes and the device will turn off automatically.
5. If the device cover is opened during the sterilization process, UV-C radiation will be stopped and the sterilization will continue as soon as the cover is closed.



- Exposure to UV-C rays is extremely harmful for both animal and human health.
- Do not expose your skin and eyes to UV light. May cause permanent discomfort.



Kavaklı Mh. İstanbul Cd. No:21
Beylikdüzü -İstanbul /TÜRKİYE

Tel : +90 212 855 82 20 (Pbx)

Fax : +90 212 855 82 21

www.ttafmedikal.com

    /ttafelectronic