



TTAF[®]

ELEKTRONİK SAN. VE TİC. A.Ş.

MINI STERILIZATION BOX
DB-015-B-UVC-1
USER MANUAL



Basic Characteristics

- UV-C Lamp Sterilization Bag provides effective sterilization thanks to UV-C rays.
- Sterilization time is 5 minutes.
- It sterilizes all surfaces of your belongings with UV-C Lamp and reflectors providing 360 ° coverage.
- Thanks to its portable design, it can be used at home, office, hospitals, workplaces, in hotels etc. ideal for use.
- Thanks to the battery feature of the device, you can use it in your desired environment.

Technical Specifications

Product Code	: DB-015-B-UVC-1
Adapter Input Value	: 100~240VAC 50/60Hz
Adapter Output Value	: 5V DC 2A
Power Consumption	: 3W
Battery Capacity	: 1200mAh
Source of light	: UV-C Lamp
UV Ray Wavelength	: 265-285nm
Working Time	: 300 sec.
Inner Dimensions	: 125x125x60mm
External Dimensions	: 150x150x89mm
Weight	: 495g

Package Included

- I. Sterilization Bag
- II. Adapter
- III. UV Protective Glasses

Operation Steps

1. Open the cover of the device and place the product you want to sterilize into the device.
2. Press and hold the on-off button inside the device for 3 seconds.
3. The device will beep 3 times.
4. The sterilization process will end at the end of 5 minutes and the device will automatically turn off with a warning sound.
5. If the device cover is opened during the sterilization process, UV-C radiation will be stopped and the sterilization will continue as soon as the cover is closed.
6. When the device is out of charge, charge it with its adapter.
7. While the device is charging, the red indicator light will be on. When the device is fully charged, the green indicator light will turn on.



- Exposure to UV-C rays is extremely harmful for both animal and human health.
- Do not expose your skin and eyes to UV light. May cause permanent discomfort.



Kavaklı Mh. İstanbul Cd. No:21
Beylikdüzü -İstanbul /TÜRKİYE

Tel : +90 212 855 82 20 (Pbx)

Fax : +90 212 855 82 21

www.ttafmedikal.com

    /ttafelectronic